


use the actuator with a canister of medicine from any other inhaler. **Do not** use ALVESCO canister with an actuator from any other inhaler. (See **Figure A**)

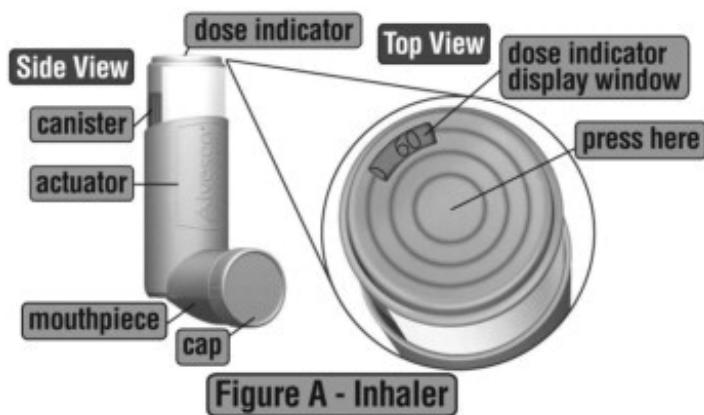

Priming your ALVESCO for use

- Remove your ALVESCO from its package.
- **Before you use ALVESCO for the first time** or if you have not used your medicine for 10 days in a row, you will need to prime your ALVESCO unit.
- Remove the plastic cap. Look at the dose indicator on top of the inhaler. Make sure that the dose indicator display window pointer is before the “60” inhalation mark before you use your ALVESCO for the first time.
- Hold the actuator upright. Spray 3 times into the air away from the face, by pressing down fully onto the center of the dose indicator button. (See **Figure B**)



- Check the dose indicator display window after the priming sprays and before the first use to make sure it shows that there are 60 sprays left in your ALVESCO unit. If there are not 60 sprays left in your ALVESCO after the first use priming spray, return it to the pharmacy.
- Make sure the canister is firmly placed in the mouthpiece each time you use your ALVESCO.

- You do not need to shake your ALVESCO unit before you use it.

Using Your ALVESCO

Step 1. Remove the cap from the mouthpiece. (See **Figure C**)

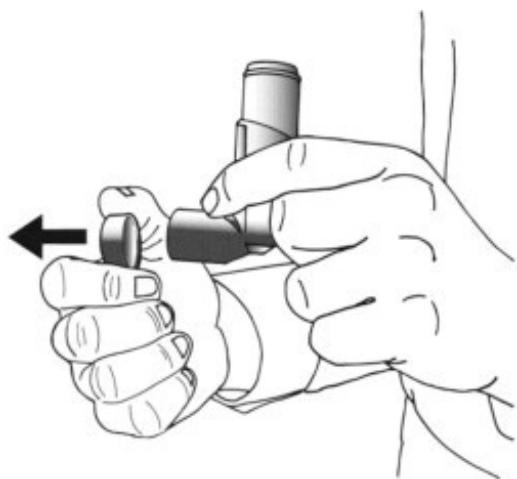



Figure C

Step 2. Hold the actuator upright, between your thumb, forefinger, and middle finger with the mouthpiece pointing towards you. (See **Figure D**)



Figure D

Step 3. Breathe out as fully as you comfortably can. Close your lips around the mouthpiece, keeping your tongue below it. (See **Figure E**)

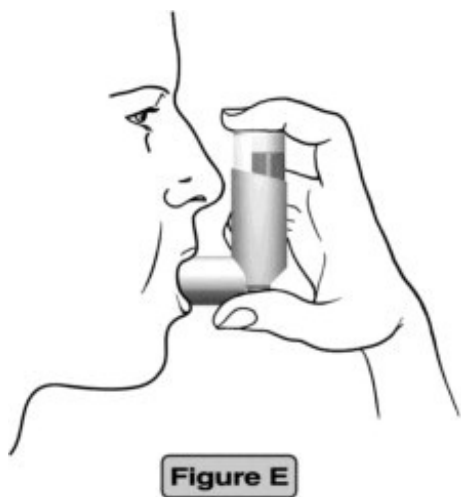

Step 4.

- While breathing in deeply and slowly, press down on the center of the dose indicator with your finger. Press down fully on the canister until it stops moving in the actuator while delivering your dose.
- When you have finished breathing in, hold your breath for about 10 seconds, or for as long as is comfortable.
- Note: It is normal to hear a soft click from the indicator as it counts down during use.

Step 5.

Take your finger completely off the center of the dose indicator and remove the inhaler from your mouth. Breathe out gently. (See **Figure F**)



Step 6. Replace the cap to keep the mouthpiece clean.

Step 7. Rinse your mouth with water and spit it out. **Do not** swallow.

Cleaning your ALVESCO unit

- Clean the mouthpiece weekly with a clean dry tissue, both inside and out. (See **Figure G**)



Figure G

- Wipe over the front of the small hole where the medicine comes out with a dry, folded tissue. (See **Figure H**)

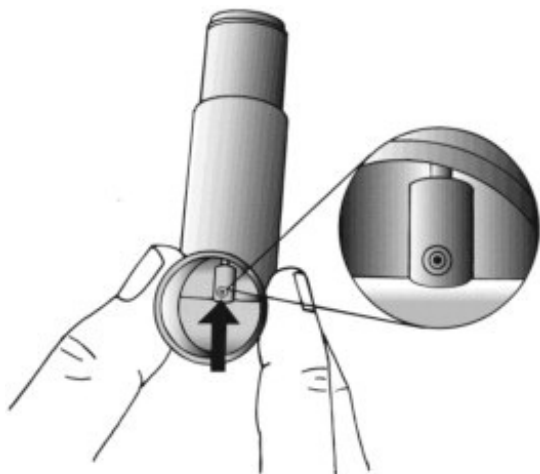


Figure H

- Do not wash or put any part of your ALVESCO unit in water or any other liquids.

How to tell if your ALVESCO canister is empty

- Your ALVESCO unit is fitted with a dose indicator display which shows you how much of your medicine is left after each use.
- Each canister of ALVESCO contains enough medicine for you to spray your medicine 60 times. This does not count the first sprays used for priming.
- The dose indicator display counts down by 10 and will move every tenth time you take a puff (i.e., 60-50-40, etc.).
- The dose indicator display window will turn red when there are only 20 sprays left. This means that you need to replace your inhaler soon.
- When the dose indicator display window reads "0" you should throw away your ALVESCO unit. (See **Figure I**)

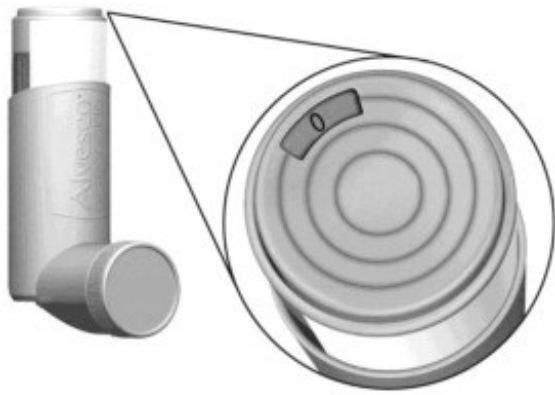


Figure 1

- Although your ALVESCO unit is fitted with a dose indicator display to help determine the number of sprays left, you should keep track of the number of sprays used from each canister of your ALVESCO unit.

This PPI and Instructions for Use has been approved by the U.S. Food and Drug Administration.

Manufactured for:

Covis Pharma

Zug, 6300 Switzerland

Made in United Kingdom

ALVESCO is a registered trademark of Covis Pharma.

©2023 Covis Pharma. All rights reserved.

PRINCIPAL DISPLAY PANEL - 80 mcg Carton